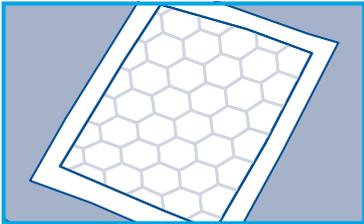


Attends® Cover-Dri

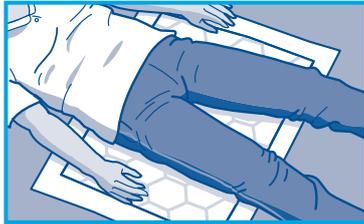


Fitting Guide

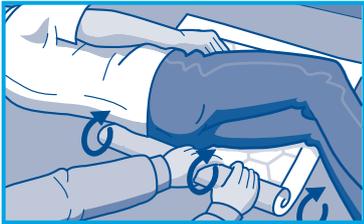
Preparation

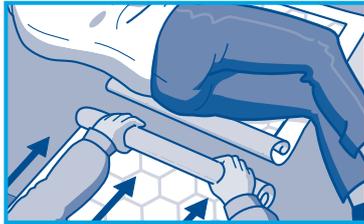
1  Open the product fully making sure that the waterproof back sheet is positioned on the bed and the soft white absorbent cushions are next to the skin.

Fitting

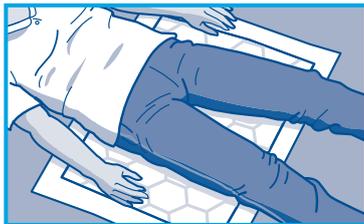
2  Position the person on the underpad.

Changing

3  Turn the person into the lateral position and roll up the used underpad towards the person's back.

4  Unfold the new underpad, positioning it towards the back of the person.

5  Turn the person on to the underpad and ensure that the pad is lying flat on the bed. Remove the used product.

6  Lay the person in their original position.

Hints and Tips

Selection

- Choose the correct product size and absorbency dependent on the clinical situation you are using the product in.

Leakage

- Check that you have the correct size and absorbency of underpad for the situation it is being used in.

Disposal

- Roll up pads and dispose as per your Trust's clinical waste policy.
- Do not flush the pad down the toilet.



Attends Cover-Dri are tested and approved by proDERM, Institute for Applied Dermatological Research.

For more information visit www.attends.co.uk